





LOWER BACK spider™

Pre-cut application instructions

Taping is an art and takes time to perfect, please practice and be patient while applying Spider Tech applications until you are familiar with the handling of the tape.

Remember to:

- Tear all perforations before applying
- Do not touch the adhesive side of the tape
- Do not completely remove the backing of the tape until 90% of the tape is adhered to the skin
- Ensure each section is applied before moving on to the next section
- Rub over the top of the tape to activate the glue

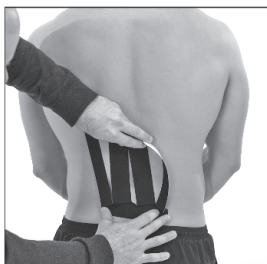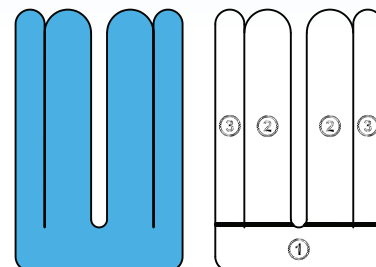


1

With the client standing, peel the backing half off of section 1 and apply to the base of the lower back. Once applied, peel the remaining portion of section 1 off and adhere to the skin. Gently rub over top of the tape to activate the glue.



2

Have the client bend forward and support their upper body with their arms if needed.

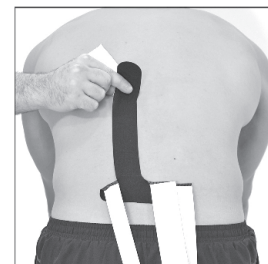

3

Start peeling the backing off of the left "arm" of section 2, and with no stretch on the tape, apply along the muscles on the left side of the spine. As you peel off the backing of the tape, allow the tape to contact the skin. Continue until there is approximately one inch left and then tear the backing off completely allowing the tape to adhere to the skin without any tension.



4

Peel the backing off of the right arm of section 2, and with no stretch on the tape, apply along the muscles on the right side of the spine as you peel off the backing of the tape until there is approximately one inch left and then tear the backing off completely allowing the tape to adhere to the skin without any tension.



5

Peel the backing off the right arm of section 3, and with no stretch on the tape, apply in the direction of the right armpit as you peel off the backing of the tape until there is approximately one inch left and then tear the backing off completely allowing the tape to adhere to the skin without any tension.



6

Peel the backing off the opposite arm of section 3, and with no stretch on the tape, apply in the direction of the left armpit as you peel off the backing of the tape until there is approximately one inch left and then tear the backing off completely allowing the tape to adhere to the skin without any tension.



Produced by:
NUCAP Medical Inc.
3370 Pharmacy Avenue,
Toronto, Ontario
Canada M1W 3K4

Tel: (416) 490-4000
Fax: (416) 490-4001

info@nucapmedical.com
www.nucapmedical.com

For complete instructions visit
www.nucapmedical.com



Patents and design patents pending - FOR PROFESSIONAL USE ONLY

All SpiderTech™ products use

The original Kinesiology Tape from Japan

